

BioDrop MD

Preservative-free, sterile, moisturizing and lubricating eye drop with 0.2% hyaluronan

What this is and what it is intended for

This is moisturizing eye drop intended to relieve mild and moderate dry eye symptoms. The moisturizing and protective properties of hyaluronan relieve irritation, dryness and burning sensations in the eyes caused by environmental elements such as dry air, wind, smoke, air pollution, excessive light, visual display work or ocular surgery and contact lens use. The composition of the eye drop helps to stabilize the tear fluid and, because the eye drop is preservative-free, it is suitable for sensitive eyes and for frequent use. Can be used before and during contact lens wear.

Instructions for use

Tear off the seal of the cap to open the bottle for the first time. Open the cap. Practise handling of the bottle before the first application. Squeeze the bottle only in tip down position. Apply 1–2 drops in the eyes 2–3 times daily, as needed. You may also apply 1 drop on contact lens before inserting it on the eye and as needed during contact lens wear. Close the cap immediately after use. The contents should be used within six (6) months from opening and by the expiry date.

Precautions and warnings

- Do not use if you are sensitive to any of the ingredients.
- Do not use if the bottle is damaged.
- The bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with fingers.
- Do not squeeze the bottle with cap on.
- Do not use after the expiry date shown on the bottle or after discard date (6 months from opening).
- Mild temporary blurring of vision may be experienced immediately after application. It is therefore advisable to wait a few seconds before continuing any activity that requires clear vision.
- Leave at least 10–15 minutes between applications of possible other ophthalmic preparations.
- Keep out of the reach and sight of children.
- Store at 5–30 °C, protect from direct sunshine and freezing.
- Discard the empty bottle appropriately.

Composition

0.2% hyaluronan in sterile isotonic, low-phosphate buffer solution. No materials of animal origin.

Last revision: September 2018



Oy Finnsusp Ab, Pääskykalliontie 5, 21420 Lieto, Finland www.finnsusp.fi

