

### What is this and what is it intended to?

This eye spray is intended to relieve dryness and irritation in the eyes, the eyelids, and the skin around the eyes. Always apply the spray on closed eyelids only. The emulsion contains sea buckthorn seed oil that slowly spreads on the tear film. The oil strengthens the tear lipid layer to protect the eyes from drying. Also, its natural antioxidant compounds protect the eye surface and the skin against cell damage and speed up their healing. Hyaluronan supports the water binding capacity of the tear film and lubricates the eyes; it also prevents cell damage and moisturizes the surrounding skin. Thanks to its special design, the bottle can be used in any position. Developed and manufactured in Finland, this eye spray is a registered Medical Device product with no alcohol, no preservatives, and no ingredients of animal origin.

### Instructions for use

Take off the protection cap. Before the first application, test the spraying mechanism a few times on your arm or in the air. Notice that there is a small step in the nozzle; to apply the spray, press the nozzle fully down. Next, hold the bottle approximately 25 cm (10 inches) from the eyes. Close both eyes and then apply 1–2 sprays on the eyelids. Once you are finished put the protection cap bac on the nozzle head. You may use the spray several times a day and during contact lens wear. If you are using other personal care products or make-up in the eye area, let them settle for at least 10 minutes before applying the spray. Or, use the other product not earlier than 10 minutes after applying the spray. Use within three (3) months from opening the bottle and by the expiry date indicated in the stamp on the bottle.

### Precautions and warnings

- No specific contraindications. Do not use if you are sensitive to any of the ingredients.
- Do not use if the bottle is damaged.
- To avoid contamination, do not touch the eye or skin with the tip of the nozzle.
- Do not touch the tip of the nozzle with fingers.
- If the spray gets directly in the eye, it may cause temporary uncomfortable feeling. If needed, rinse the eyes with clean water.
- Leave at least 10 minutes between applications of the spray and an ophthalmic medication.
- Keep out of reach and sight of children.
- Store at 5–25 °C, protect from freezing, direct sunshine and heat. Do not use the product in the event it has frozen or it has been stored in hot conditions for a long time.
- Discard the empty bottle appropriately. Due to intentional overfill, a small volume of the solution will remain in the bottle.

**Composition:** Sea buckthorn seed oil, hyaluronan and glycerol in a citrate-borate buffered emulsion.

**Latest revision:** November 2017



Finnsusp Ltd, Pääskykalliontie 5, FI-21420 Lieto, Finland.

