

What it is and what it is intended to

Piiloset BioTwin® multipurpose care solution is for cleaning, disinfecting, storing, rinsing and moisturising all contact lenses and especially silicone hydrogel contact lenses. The innovative double active Twin-EdaXyl® lubricates the contact lenses and, in daily use removes impurities from the lenses. Biotechnically produced hyaluronan moisturises contact lenses and eyes. The care solution is manufactured in Finland and developed in co-operation with the Allergy and Asthma Federation.

Instructions for use

Clean, rinse and disinfect your contact lenses daily to prevent microbes and impurities from attaching on to the lenses.

1. Wash, rinse and wipe your hands carefully.
2. Remove lens from the right eye and place it on the palm of your hand. Apply a few drops of BioTwin and rub the lens very gently with a finger on both sides for 10-15 seconds.
3. Apply more care BioTwin and rinse the lens carefully and place it in the lens container. Repeat the same procedure for the left lens.
4. Fill the container with fresh BioTwin up to the mark. Leave the lenses in the container for disinfection for a minimum of 4 hours or overnight. Always rinse your contact lenses with fresh care solution before wearing them. When taking the lenses in use, empty the container, rinse it with fresh BioTwin and let dry. Never re-use the solution.

Important

Contact lenses can be safely used only after consulting a contact lens specialist or an ophthalmologist. Follow these instructions for use and your specialist's recommendations carefully in order to guarantee safe and appropriate use of contact lenses.

- If irritation or persistent eye discomfort occurs, remove your contact lenses immediately and consult your contact lens specialist.
- Remove the safety ring under the screw cap when opening the bottle for the first time. Do not use the solution if the safety ring or bottle is damaged.
- Do not touch the tip of the bottle in order to prevent impurities in the solution.
- Never re-use the solution.
- Close the bottle carefully after use and store at room temperature (15-30 °C).
- Never mix this solution with other care solutions. It is not recommended to change a care solution brand during the replacement period of contact lenses.
- Never rinse your contact lenses or lens case with tap water.
- If the lenses have not been used for more than two weeks, disinfect them with fresh care solution before use (see step 4).
- Make sure the lens case is always clean. Rinse the case with fresh solution and let it dry open upside down. Replace the lens case with a new one regularly.
- Keep out of the reach of children.

Contraindications

Do not use if you are allergic to any ingredient. If you are allergic to silver, do not use this multipurpose solution's specific lens container. Protect the bottle from freezing and direct sunshine.

Shelf-life

The product must be used within four (4) months of opening the bottle and by the expiry date (see the stamp on the bottle). If there is solution left in the bottle after 4 months, discard it.

Ingredients

DiSodium Edetate, Twin-EdaXyl®, Hyaluronan, Polyhexanide 0.0001% in sterile isotonic and buffered solution.

Manufacturer

Finnsusp Ltd, Pääskykalliontie 5, FI-21420 Lieto, Finland, www.finnsusp.fi

Last revision: 11/2018