

### What this is and what it is intended to

This eye spray is intended to relieve dryness and irritation in the eyes, the eyelids, and the skin around the eyes. Always apply the spray on closed eyelids only. The emulsion contains sea buckthorn seed oil that slowly spreads on the tear film. The oil strengthens the tear lipid layer to protect the eyes from drying. Also, its natural antioxidant compounds protect the eye surface and the skin against cell damage and speed up their healing. Hyaluronan supports the water binding capacity of the tear film and lubricates the eyes; it also prevents cell damage and moisturizes the surrounding skin. Thanks to its special design, the bottle can be used in any position. Developed and manufactured in Finland, this eye spray is a registered Medical Device product with no alcohol, no preservatives, and no ingredients of animal origin.

### Instructions for use

Take off the protection cap. Before the first application, test the spraying mechanism a few times on your arm or in the air. Notice that there is a small step in the nozzle; to apply the spray, press the nozzle fully down. Next, hold the bottle approximately 15–20 cm (6–8 inches) from the eyes. Close both eyes and then apply 1–2 sprays on the eyelids. Once you are ready, put the protection cap back on the nozzle head. You may use the spray several times a day and during contact lens wear. If you are using other personal care products in the eye area, let them settle for at least 10 minutes before applying the spray, or, use the other product not earlier than 10 minutes after applying the spray. If you are using eye make-up, you may wish to apply the spray from a longer distance to avoid spreading, depending on the type of make-up. Use within six (6) months from opening the bottle and by the expiry date indicated in the stamp on the bottle.

### Precautions and warnings

- Do not use if you are sensitive to any of the ingredients.
- Do not use if the bottle is damaged.
- To avoid contamination, do not touch the eye or skin with the tip of the nozzle.
- Do not touch the tip of the nozzle with fingers.
- If the spray is applied on an open eye, you may temporarily experience mild discomfort
- Leave at least 10 minutes between applications of the spray and an ophthalmic medication.
- Keep out of reach and sight of children.
- Store at 5–25 °C, protect from direct sunshine and heat. Do not use the product in the event it has frozen or it has been stored in hot conditions for a long time.
- Discard the empty bottle appropriately. Due to intentional overfill, a small volume of the solution will remain in the bottle.

**Composition:** Sea buckthorn seed oil, hyaluronan and glycerol in a sterile hypotonic trometamol/citrate buffer emulsion.

**Latest revision:** 10/2019



Finnsusp Oy, Pääskykalliontie 5, FI-21420 Lieto, Finland. [www.finnsusp.fi](http://www.finnsusp.fi)

