

WHAT THIS IS AND WHAT IT IS INTENDED TO

This all-in-one product is a care solution regimen for cleaning all soft and silicone hydrogel contact lenses as well as for their protein removal, disinfection, storage and rinsing. A biological cleaning agent that also removes proteins and lipids in daily use improves the disinfectant effect of the solution.

INSTRUCTION FOR USE

Clean, rinse and disinfect your contact lenses daily to prevent microbes and impurities from attaching to the lenses.

1. Wash, rinse and dry your hands carefully.
2. Remove the right lens from the eye and place it on the palm of your hand.
3. Apply a few drops of the all-in-one and rub the lens very gently with a finger on both sides for 10-15 seconds.
4. Apply more of the all-in-one product and rinse the lens carefully. To reach the best result, repeat the procedure. Place the lens in the lens container. Repeat procedure for the other lens.
5. Fill the container with fresh all-in-one solution up to the mark. Leave the lenses in the container for disinfection for a minimum of 4 hours or overnight. Always rinse your contact lenses with fresh solution before wearing them. When taking the lenses into use, empty the container, rinse it with fresh care solution, and let dry.

IMPORTANT

Contact lenses can be safely used only after consulting a contact lens specialist or an ophthalmologist. Follow these instructions for use and your specialist's recommendations in order to guarantee safe and appropriate use of contact lenses.

- If irritation or persistent eye discomfort occurs, remove your contact lenses immediately and consult your contact lens specialist.
- Remove the safety ring under the screw cap when opening the all-in-one bottle for the first time. Do not use the solution if the safety ring or bottle is damaged.
- Do not touch the tip of the bottle in order to prevent impurities in the solution.
- Never re-use the solution.
- Close the bottle carefully after use and store at room temperature (15-30 °C).
- Never mix this all-in-one with other care solutions. It is not recommended to change a care solution brand during the replacement period of contact lenses.
- Do not transfer this all-in-one into another container.
- Never rinse your contact lenses or lens case with tap water.
- If the lenses have not been used for more than two weeks, disinfect them with fresh all-in-one solution before use (see step 5).
- Make sure the lens case is always clean. Rinse the case with fresh solution and let it dry open. Replace the lens case with a new one regularly.
- This all-in-one product should be used within four (4) months of opening the bottle and by the expiry date (see the stamp on the bottle).
- Protect the bottle from freezing and direct sunshine. Keep out of the reach of children.

CONTRAINDICATIONS Do not use if you are allergic to any ingredient.

COMPOSITION Disodium EDTA, biological cleaning agent, polyhexanide 0.0001%, sodium borate and sodium chloride in sterile, isotonic and buffered solution.

Latest revision: 10/2017



Oy Finnsusp Ab,
Pääskykalliontie 5
21420 Lieto, Finland
www.finnsusp.fi

